

# See The Daily Difference



## With C·VUE 1 DAY ASV Daily Disposable Contact Lenses

C·VUE SUPERIOR TECHNOLOGY | 1 DAY DISPOSABLE | SPECIALTY SINGLE VISION

### C·VUE 1 DAY ASV

Wavefront Technology

Superb Comfort and Convenience

120 Day Performance Guarantee

Daily Disposable

Base Curve: 8.5/Diameter: 14.2

Power Range: +4.00 to -8.00

Methafilcon A 55%

Free Trial & Bank Programs

Improved Depth of Focus  
for Early Presbyopes

### A daily disposable available exclusively through licensed eye care professionals

Finally, a daily specialty lens option from Unilens, that does not require the expense and time of cleaning solutions. C·VUE 1 DAY ASV is an aspheric single vision solution that features a Wavefront technology applied design, which results in the maximum aberration control possible in a molded disposable lens. Offer your patients a daily disposable, single vision lens that features improved visual acuity at all distances, especially in moderate to high prescriptions. Ideal for those new to contact lens wear or those who have an active lifestyle.

**Unilens**  
The Eye Care Professional's Specialty Lens Company

[www.unilens.com](http://www.unilens.com) • 800-446-2020

# C·VUE® 1 DAY ASPHERIC SINGLE VISION

(methafilcon A) Soft (hydrophilic) Daily Disposable Contact Lens

## Fitting Guidelines

### SUGGESTED PATIENT CRITERIA:

- Normal binocular vision and good ocular health
- Refractive astigmatism below 0.75D

### LENS SELECTION:

#### STANDARD PARAMETERS:

Base Curve/Diameter: 8.5/14.2

Powers: +4.00D to -8.00D

- 0.25D increments from +4.00D to -6.00D
- 0.50D increments from -6.50D to -8.00D

#### INITIAL FIT DETERMINATION:

- Select the appropriate trial lens based on the patient's spectacle prescription in minus cylinder form.
  - Ignore any cylinder and compensate for vertex distance if necessary.
- Allow lenses to equilibrate for 20 minutes.
  - Lens comfort should be acceptable after equilibration.
  - Lens should center well with 0.5mm to 1.0mm eye movement in primary gaze and 1.0mm to 1.5mm upward gaze.
- Measure acuity in normal room illumination.
  - Perform an over-refraction.
  - Add any compensated power to the trial lens power to obtain the corrected lens to be ordered.

### VISUAL ACUITY UNACCEPTABLE:

- Perform over-refraction compensating for vertex distance. Add the over-refraction to the lens power to obtain the new lens power to be ordered.

### THE NEW C·VUE® 1 DAY ASV



**Unilens**  
The Eye Care Professional's Specialty Lens Company  
[www.unilens.com](http://www.unilens.com)

TO PLACE AN ORDER OR FOR CONSULTATION  
CALL 1-800-446-2020

CAUTION: Federal law restricts this device to sale by or on the order of a licensed practitioner.  
For additional important information call Unilens at 1-800-446-2020 for a detailed C·VUE 1 DAY Aspheric Single Vision  
(methafilcon A) Soft (hydrophilic) Contact Lens Professional Fitting Guide.